



**SF338**

### **ARCHITECTS' AND ENGINEERS' SPECIFICATIONS**

Staff emergency psychiatric station(s) shall be Alpha Communications®/Tektone SF338 or approved equal. Operation of all emergency signals shall be accomplished by depressing the centrally located push button switch designated "STAFF ASSIST". The switch shall be enabled only after activation of the key switch located on the associated IR316 psychiatric station. The red call-placed indicator on the SF338 shall flash to assure that the call has been properly placed. Call cancellation shall be possible only at the IR316 Psychiatric Station.

Reliable, solid state components shall be used and plug-in connectors shall be provided for electrical connections.

The SF338 shall be UL 1069 listed to the latest 4th Edition Standards. Contractor shall observe all local and national electrical and building codes.

Copyright© 1997-2001, Alpha Communications®, All Rights Reserved

## **SF338 Staff Emergency Psychiatric Station**

The SF338 Staff Emergency Psychiatric Station provides emergency call signaling when used in conjunction with the Alpha Communications® IR316 psychiatric station. The station is for use in psychiatric wards or other high security hospital areas and is normally mounted within the ward or area where the patients are accommodated and within easy reach of the nursing staff.

An emergency call can only be placed following activation of the key switch on the IR316 station, usually located at the entrance to the psychiatric or high security area. A call is placed by depressing the call push button. The call-placed indicator flashes to provide for call assurance. Calls are registered on the NC300 series master station (and duty station if used) via audible and visual signals, as well as local indication on dome lights. Call cancellation is possible only at the IR316 psychiatric station.

### **FEATURES**

- UL 1069 Listed - 4th Edition Standards
- Simple, reliable operation
- Call-placed indicator
- Call placed only after activation of IR316 key switch, ensuring use by authorized personnel only
- Call registers at master station, duty station and on dome lamp
- Mounted on stainless steel panel
- Call cancellation only at the IR316 Psychiatric Station

### **SPECIFICATIONS**

<b>Dimensions:</b>	Height: 4.50" (114mm) Width: 2.75" (70mm) Depth: 1.63" (29mm) Projects: 0.38" (10mm) from mounting surface
<b>Construction:</b>	Reliable components mounted on a rugged stainless steel faceplate
<b>Construction:</b>	Plug-in connectors
<b>Housing:</b>	Steel City #52171 box with Steel City #52C14 plaster ring, or Alpha Communications® Model #IH358 box with IH357 ring, or EXACTEQUAL. Vertical mount.

*Due to continuous product improvement, all colors, sizes, materials, finishes and specifications are subject to change without notice.*